



INTERNATIONAL

# FORM FIT<sup>®</sup>

## HYDROGEL INTRACANALICULAR PLUGS


### ONE SIZE. ONE SOLUTION.

Form Fit is truly a one size plug solution. Once inserted into the canaliculus, Form Fit hydrates over a 10 minute period. As it hydrates, Form Fit increases in size until it completely fills the vertical canalicular cavity. Patent # 5,116,37.




### PRODUCT FEATURES


- Hydrates over a period of 10 minutes
- Made from hydrogel material
- Requires no special storage
- Easily removed via irrigation of the tear duct
- Biocompatible, no foreign body sensation
- Conveniently preloaded in inserter
- 0.3mm, one size fits all

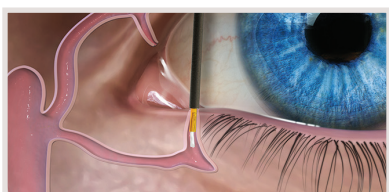

Insertion

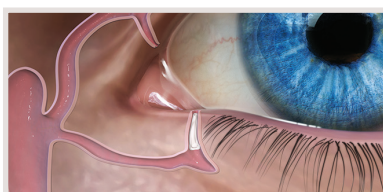

Immediately after insertion

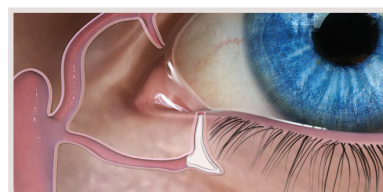


10 minutes after insertion

Product Name	Qty	Description	Ref #
FORM FIT Intracanalicular Plug 10pk	10	One size	6303
FORM FIT Intracanalicular Plug 2pk	2	One size	6303-T

## SOFT PLUG<sup>®</sup> EXTENDED DURATION INTRACANALICULAR PLUGS

Extended Duration absorbable plugs are designed to provide patient comfort up to 3 months. This long-lasting plug is effective and economical for transient dry eye symptoms after surgery. Extended Duration plugs are 2mm in length and sterile.

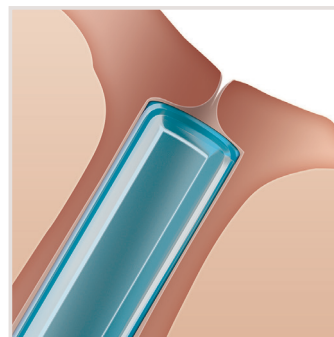


Product Name	Qty	Size	Ref #
SOFT PLUG Extended Duration Plugs 20pk	20	0.2mm	6402-A
SOFT PLUG Extended Duration Plugs 20pk	20	0.3mm	6403-A
SOFT PLUG Extended Duration Plugs 20pk	20	0.4mm	6404-A
SOFT PLUG Extended Duration Plugs 20pk	20	0.5mm	6405-A
SOFT PLUG Extended Duration Plugs 40pk	40	0.2mm	6402
SOFT PLUG Extended Duration Plugs 40pk	40	0.3mm	6403
SOFT PLUG Extended Duration Plugs 40pk	40	0.4mm	6404
SOFT PLUG Extended Duration Plugs 40pk	40	0.5mm	6405
SOFT PLUG Extended Duration Plugs Variety Pack	20	1 pair 0.2mm, 3 pairs 0.3mm, 4 pairs 0.4mm, and 2 pairs 0.5mm	6400V

# SOFT PLUG® SILICONE PUNCTUM PLUGS

The distinctive low-profile cap combined with a softer, more flexible silicone maximizes patient comfort. The inserter provides greater control and eliminates accidental release of the plug. Soft Plug silicone plugs are removable.

- Available sterile, preloaded on inserter in 2-pack or 6-pack dispenser box.
- Available in sterile, non-loaded Economy Pack with 2 inserters and 10 trays of 2 plugs each.

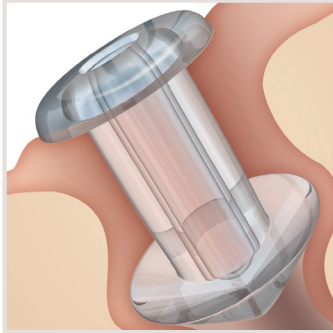


Product Name	Qty	Description	Size	Ref #
SOFT PLUG Preloaded Silicone 2pk	2	Micro	0.4mm	6610
SOFT PLUG Preloaded Silicone 2pk	2	Mini	0.5mm	6611
SOFT PLUG Preloaded Silicone 2pk	2	Petite	0.6mm	6612
SOFT PLUG Preloaded Silicone 2pk	2	Small	0.7mm	6613
SOFT PLUG Preloaded Silicone 2pk	2	Medium	0.8mm	6614
SOFT PLUG Preloaded Silicone 6pk	6	Micro	0.4mm	6610-D
SOFT PLUG Preloaded Silicone 6pk	6	Mini	0.5mm	6611-D
SOFT PLUG Preloaded Silicone 6pk	6	Petite	0.6mm	6612-D
SOFT PLUG Preloaded Silicone 6pk	6	Small	0.7mm	6613-D
SOFT PLUG Preloaded Silicone 6pk	6	Medium	0.8mm	6614-D
SOFT PLUG Silicone Economy Pack	20	Micro	0.4mm	6610-EP
SOFT PLUG Silicone Economy Pack	20	Mini	0.5mm	6611-EP
SOFT PLUG Silicone Economy Pack	20	Petite	0.6mm	6612-EP
SOFT PLUG Silicone Economy Pack	20	Small	0.7mm	6613-EP
SOFT PLUG Silicone Economy Pack	20	Medium	0.8mm	6614-EP



## SOFT PLUG® FLOW CONTROL PLUGS

- 2 plugs per box
- Silicone
- Preloaded on inserter
- For partial occlusion with limited tear drainage

Product Name	Qty	Description	Size	Ref #
SOFT PLUG Flow Control 2pk	2	Petite	0.6mm	6612-FC
SOFT PLUG Flow Control 2pk	2	Small	0.7mm	6613-FC
SOFT PLUG Flow Control 2pk	2	Medium	0.8mm	6614-FC

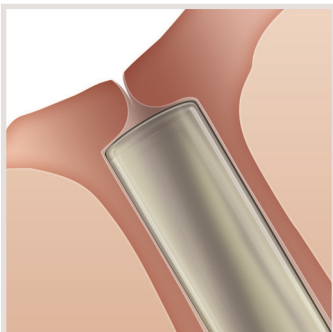


## SOFT PLUG® COLLAGEN INTRACANALICULAR PLUGS

NOT CE MARKED

Smooth proprietary finish that simplifies insertion of the plug. Ideal for short-term, diagnostic, and postsurgical occlusion. Absorbable within 2-5 days. 2mm length, sterile.

Product Name	Qty	Size	Ref #
SOFT PLUG Collagen Plugs	60	0.2mm	6502
SOFT PLUG Collagen Plugs	60	0.3mm	6503
SOFT PLUG Collagen Plugs	60	0.4mm	6504



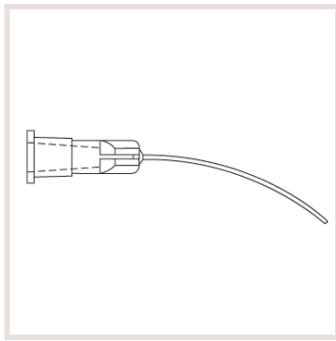

## LACRIMAL IRRIGATION KIT

Product Name	Description	Qty	Ref #
Lacrimal Irrigation Kit	One 3ml syringe and one 25g x 1 1/8" lacrimal cannula, curved, open end. For use with BSS to irrigate the Form Fit plug through the nasal passage if removal becomes necessary.	10 per box	4003-IK



## SOFT PLUG® GAUGE SET

For accurate sizing of the puncta before insertion of the plug. Sterile.

Product Name	Description	Ref #
Sizing Gauge	.5mm and .6mm tips	6625
Sizing Gauge	.7mm and .8mm tips	6627
Sizing Gauge Set	Includes both instruments and sterilization tray	6630



## DRY EYE INSTRUMENTS

Product Name	Description	Ref #
Straight Jeweler's Forceps	Straight #3, sterile	6643
Curved Jeweler's Forceps	Curved #7, sterile	6647
Grooved Forceps	For the insertion of Extended Duration & Collagen Plugs, sterile	6648
Castroviejo Lacrimal Dilator	Double Ended, 14cm, sterile	6640

### OUR SOLUTION BASED PRODUCTS

Our values of integrity and quality guide our commitment to solution-based products that enhance the patient care. From the beginning, OASIS has been a practice-based company focused on building the Dry Eye and Surgical practice through the various tools we offer.



**Donawa Lifescience Consulting Srl**  
Piazza Albania 10  
00153 Rome, Italy



Glendora, CA  
international@oasismedical.com  
(909)305-5400 | www.oasismedical.com



EN ISO 13485:2012 + AC : 2012  
LIT-CM006E 1/15