



COD. 100401 - EASYTON PLUS BOX 100 PCS

Disposable prism for applanation tonometry

- HIGH DEFINITION MEASURE
- PRACTICAL
- DISPOSABLE
- SAFE
- EASY
- STERILE



PRACTICAL

Everything you need for a complete sterile tonometry in one pack.

EASY

The tonometry prism fits easily on all types of Goldmann tonometers.

STERILE

Prevents risks of infections.

DISPOSABLE

No need to clean the tonometer prism. Simply throw it away. Your patients will be impressed with your attention to hygiene. You will be impressed with its speed of use.

SAFE

Sterilised with ethylene oxide, the tonometer prism is safe to use with no risk of allergic reaction or infection.


Each individual EASYTON blister contains a sterile tonometer prism tip. This is all that you need to perform an accurate and sterile tonometry procedure. For your economical convenience, EASYTON comes in an attractive dispenser holding 100 pieces.



Remove the tonometer prism from the sterile blister careful not to contaminate the tip.



Insert the tip into the tonometer with the line aligned to the zero position.

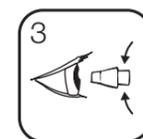

Perform the intraocular pressure measurement.



Dispose the used tonometer tip.

CE 0476

Patented

MADE IN ITALY