


Floor Stand with Casters for ESV3000™, ESC2000™ and CSV1000™

Part Number: 500004

ESV3000™, CSV1000™, or ESC2000™ Base with Casters

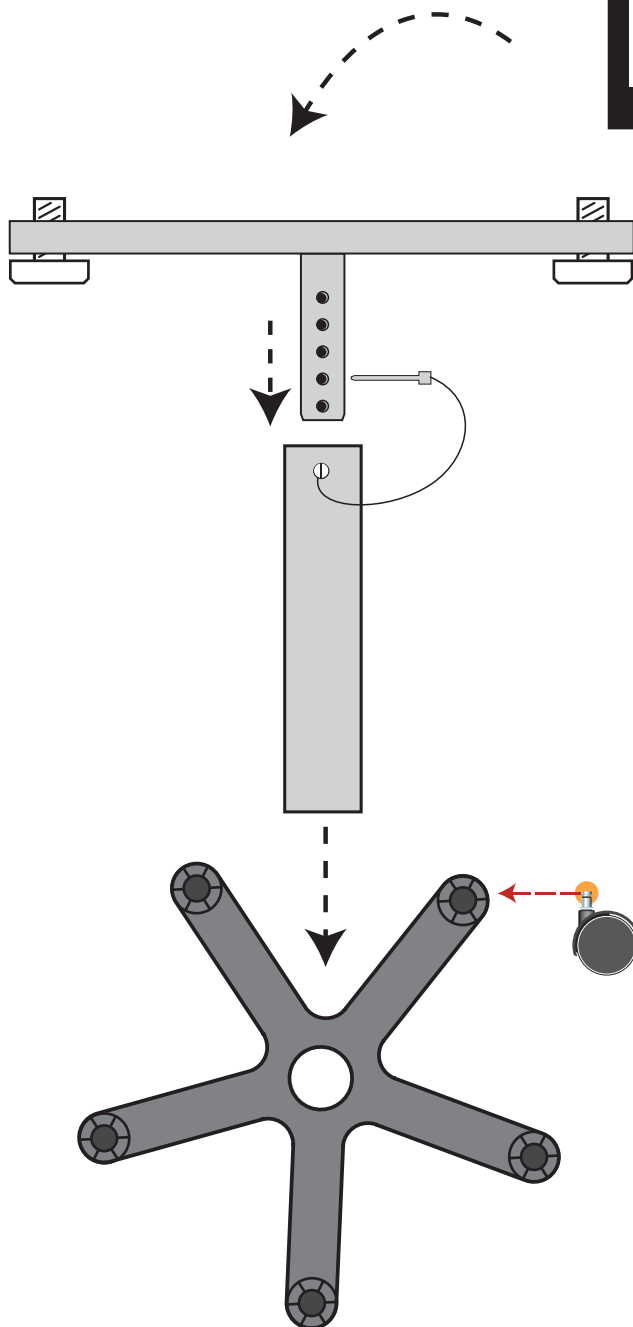

x4

The ESC2000™ uses these 2 holes



The ESV3000™ and CSV1000™ use these 4 holes



1. Insert casters (5) into the five-legged base.
2. Insert square pole into base and thread screw from underneath the base for a secure connection.
3. Flip illuminator cabinet upside down to expose the 2 (or 4) mounting holes.
4. Place T-Bar on the cabinet, lining up the corresponding mounting holes on the illuminator. Thread each screw tightly on each side.
5. Lift cabinet and attached T-bar, flip illuminator cabinet right side up and guide the circular pole into the square pole.
6. Insert push pin into circular pole to desired height. Make sure to push pin holder through both holes on the T-bar pole.

Floor Stand with Casters for ESV3000™, ESC2000™ and CSV1000™

Part Number: 500004



1155 Jansen Farm Drive
Elgin, IL 60123 USA

Toll Free: 800-362-3860 Phone: 847-841-1145
Toll Free Fax: 888-362-2576 Fax: 847-841-1149

www.good-lite.com




Good-Lite B.V.
Kantstraat 19, NL-5076NP Haaren
P.O. Box 10110, NL-5060 GA Oisterwijk
The Netherlands
EAB@medical-risk.com
+31 (0) 411.623.791
SRN: NL-AR-000002695



QNET Ltd.
Livingstone House, 309 Harrow Road
Wembley, Middlesex HA9 6BD (GB)
UKRP@medical-risk.com
MHRA Account No: 0000011991



QNET CH-REP GmbH
Im Büel 15
8750 Glarus (Switzerland)
CH-REP@medical-risk.com
UID Number: CHE-313.448.831
Ph.: +41 79 610 0870

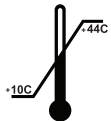

MD Floor Stand with Casters for ESV3000™, ESC2000™ and CSV1000™

REF 500004